



TRANÅS ROSTFRIA

TR 1700 HILO, FREE-STANDING HEIGHT ADJUSTABLE BATHTUB INSTALLATION INSTRUCTIONS, see DRAWING for measures.

PLUMBING ROUGH - IN

The bathtub is freestanding and with adjustable feet.

Drainage	100 litre per minute. To be connected to floor or wall.
Weight empty	95 kg
Water volume	160 lit

Option: Factory mounted CORALLO thermostat.

Water pressure 1 - 5 bar. Inlet dynamic pressures nominally balanced to within 10% of each other during flow.
A pressure reducer is recommended when more than 5 bar.
Adjust the thermostat when more than 6 bar difference on hot and cold water.

Hot water temp Inlet water temp min. 65 C. Differential of approx. 50 degrees between hot and cold water inlet temperatures.

Thermostat Factory set at 38 C, calibrated at 3 bar on both sides.

Flow rate 15-35 l/min.

To calibrate the thermostat see the enclosed manual.

Avoid any kinking curves when bending the hot and cold water installation hoses.

To eliminate pipe debris it is essential that supply pipes are thoroughly flushed before connected to the bathtub hoses.

Pipes, hoses and fittings, to and from bath connections including controllable backflow preventors (checkvalves), shut off valves and filters, to be supplied and installed by others and to comply with local building and plumbing codes.

ELECTRICAL ROUGH-IN

Standard electrical requirements: 230 volts, 50 Hz, 6 Amps. IP65.



NOTE: All electrical conduits, fittings and wires to bath supplied and installed by your professional electrician in accordance with local or applicable electric code as a minimum requirement. Permanent connection.

Main switch with signal lamp, main fuse and earth circuit breaker are not included and must be supplied by others.

CAUTION: Polarity and grounding must be correct. Improper connection may result in injury.

June 2005

TRANÅS ROSTFRIA AB
PO BOX 116, SE-573 22 TRANÅS, SWEDEN
PHONE +46 (0)140-38 50 60 FAX+46 (0)140 38 50 61
e-mail:info@trequipment.se
www.trequipment.se

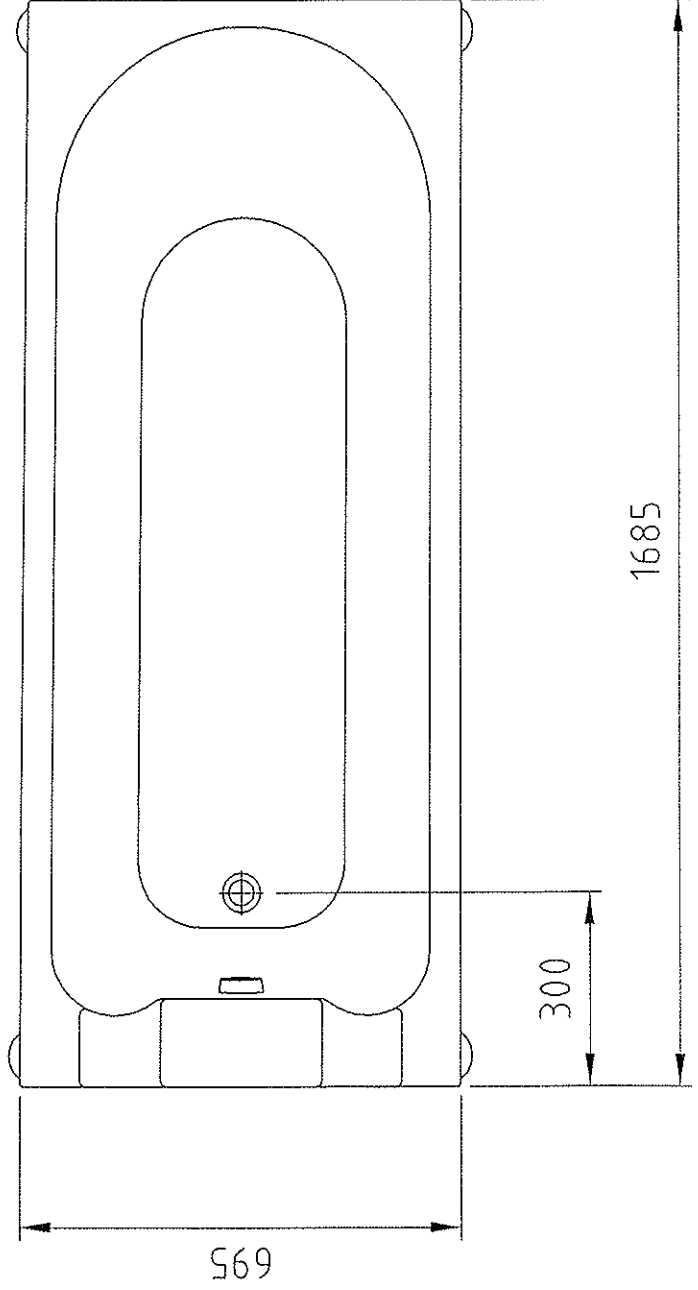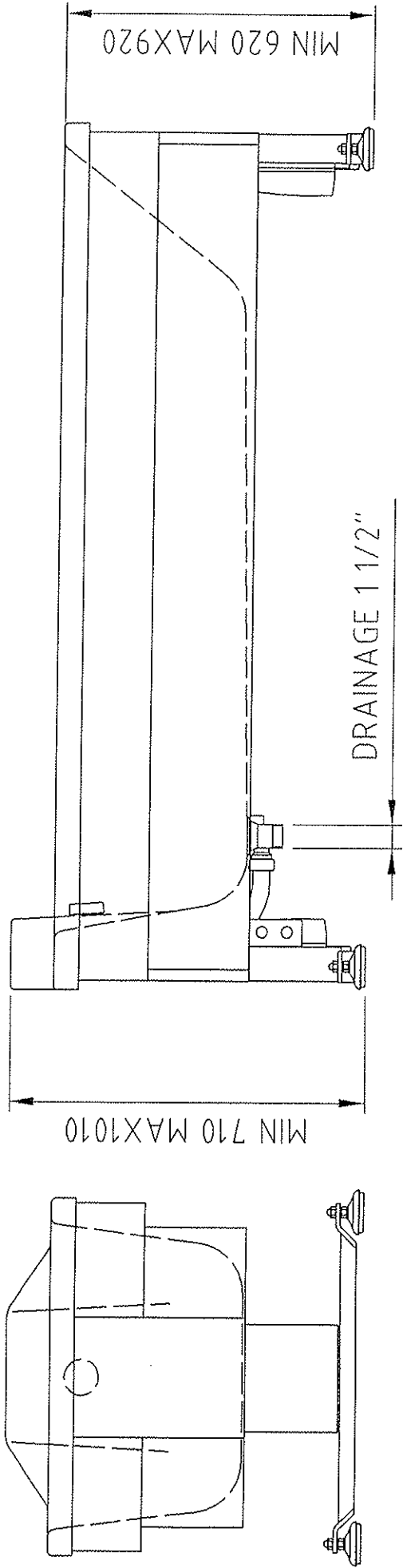

TRANÁS ROSTFRIA

**TR 1700 HILO
INSTALLATION DRAWING**