

INSTRUCTIONS FOR USE

CRADLE TOILET SLINGS HC01-1200, HC01-1250, HC01-1400, HC01-1450

Several different styles of slings are available for use with the Joey, Jumbuck and Wombat Patient Lifters with cradle frame, all attach to the lifters in the same way. The Toilet Cradle Slings are designed for maximum client comfort, support and safety using materials of the highest quality available. All padded areas are of a closed cell, water repellent, and chemically inert foam. Thus ensuring the padding will not harbour any bacteria and/or moisture and will not be affected by urine or detergents. The slings are fully washable and durable.

All slings are tested to International Standards ISO/DIS 10535 with a Safe Working Load **S.W.L of 240 kilos**

General Instructions from sitting position:

- 1 Partially lean the client forward, so that the sling can be slid down back.
- 2 Position centre of sling at centre of clients back, ensuring the pads are under the clients arms, (for HC01-1200/50 top head support should be just above head height)
- 3 Adjust and fasten chest strap firmly, (for HC01-1200/50 clip up head straps and adjust as appropriate.)
- 4 Alternately, lift clients legs and place leg strap under thigh area, ensure that padded section is under middle thigh and strap comes up between clients legs.
- 5 Position lifting machine in front of client, so as cradle pivot is directly over the centre of gravity of the patient, ready to accept sling attachments.
- 6 Lower the arm so the sling clips can be attached. Attach the 2 shoulder clips first.
- 7 Tilt the cradle down and attach the 2 leg clips.
- 8 Ensure all 4 clips are securely engaged,
- 9 Raise client (whilst holding the cradle handle) up slightly, so that sling just takes clients weight but is not yet fully off seat. Check that sling is flat against the body, chest strap is firm and that client is comfortable and safe.
- 10 The patient may now be transferred to required position.

Instructions from a bed.

- 1 Roll the client onto their side.
- 2 Bunch sling up longways and place behind the client.
- 3 Position centre of sling at centre of clients back, ensuring the pads are under the clients arms, (for HC01-1200/50 top head support should be just above head height)
- 4 Roll the client onto their other side and pull the sling through, return onto back, there should be 1 shoulder clip and 1 leg clip either side of the patient.
- 5 Adjust and secure chest strap ensuring arms are on the outside of the sling with padded section under arms.
- 6 Adjust and secure head straps for HC01-1200/50
- 7 Position lifting machine in front of client, so as cradle pivot is directly over the centre of gravity of the patient, ready to accept sling attachments.
- 8 Lower the arm so the sling clips can be attached. Attach the 2 shoulder clips first.
- 9 Alternately, lift clients legs and place leg strap under thigh area, ensure that padded section is under middle thigh and strap comes up between clients legs. Attach the 2 leg clips.
- 10 Ensure all 4 clips are securely engaged,
- 11 Raise client (whilst holding the cradle handle) up slightly, so that sling just takes clients weight but is not yet fully off seat. Check that sling is flat against the body, chest strap is firm and that client is comfortable and safe.
- 12 The patient may now be transferred to required position.

Please Note:

For more detailed user instructions for the lifter and sling use, please read the operating manual supplied with the lifter.

Important Note:

- ◆ Always use slings with label and care instructions facing outwards, so that correct side of sling is used.
- ◆ Labels must not be removed from this product.
- ◆ Always store slings with Velcro straps closed to prevent catching and piling of material and the Velcro hook from clogging.
- ◆ Always position centre of sling at centre of clients back.
- ◆ The correct sling size is important to ensure a safe lift. If unsure of what size to use always try small first
- ◆ **CLIENTS MUST NOT BE LEFT UNATTENDED WHILST IN SLING.**
- ◆ **THIS SLING MUST BE ASSESSED AS APPROPRIATE FOR THE CLIENT BY AN APPROPRIATELY QUALIFIED PERSON**
- ◆ **THIS SLING CAN ONLY BE USED ON A HOIST WITH A CRADLE FRAME.**
- ◆ Regularly check stitching on this item. If repairs required **DO NOT USE** and contact supplier immediately.
- ◆ With average use and washing, **slings must be replaced after 5 years** from manufactured date (batch number)
- ◆ Always check **Safe Working Load** of both lifter and sling, use lesser of the two as **maximum** Safe Working Load
- ◆ **If Sling label is unreadable, DO NOT USE SLING, Sling Must Be Replaced. Contact Supplier Immediately.**

BINDING / SIZE COLOUR CODES

XSmall	White
Small	Green
Medium	Yellow
Large	Red
XLarge	Black

WASHING INSTRUCTIONS

Using a Washing Bag is recommended
Close Velcro Straps & Buckles
Machine Wash **Max. 70°**
Mild detergent
Do not bleach
Maybe Tumble Dried Max. 70°

HAYCOMP PTY LTD